# QuickSleeper® 5

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- THE DEVICE can experience interference from other devices, even if they comply with CISPR EMISSION requirements.

- Do not adjust the length of the device's cables or repair or replace them (handpiece cable and power cable).

- The use of cables other than those specified, except for cables sold by Dental Hi Tec as replacement parts for internal components, may result in an increase in EMISSIONS or in a decrease in the IMMUNITY of the DEVICE.

- In certain cases, if it is necessary to use the DEVICE beside other devices or stacked with other devices, and if it is not possible to do otherwise, the DEVICE should be monitored to ensure it functions normally. (in the intended setting).

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### Miscellaneous information

- Unplug the power at the end of every day or in the event of a storm.
- Type BF degree of protection against electric shocks.
- The protection class of the pedal is IPX1.
- QuickSleeper® must not be used in an oxygen-rich environment.

### Warranty conditions

Dentalhitec offers a two-year warranty on QuickSleeper® from the date of purchase.

This warranty does not apply to the plastic and rotary containers.

The implementation of the warranty requires the submission of the original invoice indicating the date of purchase.

This warranty applies only where the product is used in accordance with the provisions of this document.

No warranty is due where:

- a technical intervention has been performed on the device by an unauthorised person.
- the device has been damaged by an external source (lightning, fire, water damage, negligence or accident), by improper use, or by a non-compliant installation.
- the device has been adapted to performance standards different from those indicated on the fact sheet located on the bottom of the control box.
- the purchase documents have been altered or modified.
- the serial number of the device has been made unreadable.

Application of this warranty does not, under any circumstances, constitute an entitlement to complete replacement of the device or give the right to claim damages and interest.

QuickSleeper® cannot be returned for reasons beyond the scope of the warranty clauses.

#### **This appliance must not be disposed of with household waste.**

It contains dangerous substances, which can have important consequences for the environment and human health. It must be recycled in a specific treatment facility.

If you are concerned by the European WEEE directive, please contact your distributor or Dentalhitec, who will take all of the necessary steps to enable you to have your device collected and recycled.

Declaration of conformity

We declare on our own liability that this product complies with EC regulatory provisions and that it has received EC and CSA certification.

In the interest of product improvement, Dentalhitec reserves the right to make modifications to its products without prior notice.

### Device repair

The device does not require regular maintenance.

If there is a problem, contact your authorised distributor directly or the Dentalhitec company headquarters.

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### GENERAL RECOMMENDATIONS

- To ensure good communication between the control box and wireless pedal, avoid obstacles between these two elements, especially metallic obstacles (computers, boxes).
- Electromagnetic and/or radio-frequency interference may occur between QuickSleeper® and other electronic devices. Keep radio devices (telephone, control box, etc.) at least 1.5 metres (5 feet) away from the control box, from the wireless pedal, and from the power supply.
- QuickSleeper® must only be used by a qualified dental surgeon in a hospital or dentist's surgery.
- Anesthesia requires constant monitoring by the practitioner.
- Do not leave QuickSleeper® within reach of children.
- Do not expose QuickSleeper® to:
  - the sun or leave it near a radiator or air conditioner.
  - water and chemical products.
- The power supply plug serves as a cut-off and must remain accessible.
- Always wear disposable surgical gloves when handling and using QuickSleeper®.

### PROTECTION AGAINST ELECTRIC SHOCKS

- The installation must be fitted with a circuit breaker.
- Before use, check the condition of the plug and power cable.
- Do not pull on the cable to remove the plug from the power socket.
- The connection must be made on a single-phase alternating current supply.
- The external power supply voltage must be identical to the one indicated on the data plate of the power supply.

### NEEDLE BREAKAGE

- A broken needle is the result of poor handling.
- Make sure the needle is not offcentre by turning it manually before penetration.

### RECOMMENDATIONS FOR THE USE OF

#### ANESTHESIA

- Use only 1.7 ml or 1.8 ml glass anesthetic carpules with a plastic protection film so that the carpule may be easily removed in case of breakage.
- Refer to the carpule manufacturer's recommendations for the quantities of anesthesia that can be injected per session.

#### SAFETY

- For safety reasons, do not screw on or unscrew the needle anywhere near your patients, and protect them with safety glasses.
- There is an integrated lip protector to avoid injuring the patient while the needle is rotating.

#### CONTAMINATION & DISINFECTION

- The needles and anesthetic carpules are single use and must be destroyed in an incinerator intended for this purpose; refer to the instructions from the manufacturers.
- The containers must be disinfected and sterilised before each anesthesia .
- The holder and handpiece must be disinfected after each patient (follow the disinfectant manufacturer's instructions).
- Do not leave needles unprotected.

### WARNING

Your QuickSleeper® must not be modified without the permission of Dentalhitec. Should your QuickSleeper® need to be modified, the modification must be carried out by a repairer authorised by Dentalhitec, or by Dentalhitec's technical service. Checks and appropriate testing must be performed to ensure that your device can still be used safely. QuickSleeper® must not be attached to other devices. Such use is the responsibility of the user.

### CONTRAINDICATIONS

There are no known contraindications to date.

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